


RSL 5000

SLIT LAMP

OPERATOR MANUAL

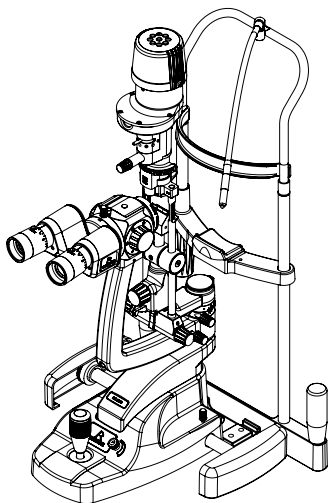






















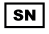
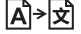

H Series



RODENSTOCK

CONTENTS

1. INDICATIONS FOR USE	3
2. SAFETY.....	3
2.1 PHOTOTOXICITY	3
2.2 WARNINGS AND CAUTIONS	4
2.3 CONTRAINDICATION.....	5
3. CLEANING AND DISINFECTION INSTRUCTIONS.....	5
4. NAMES OF CONTROLS AND COMPONENTS	6
5. ASSEMBLY	7
5.1 ILLUMINATION TOWER ASSEMBLY PROCEDURE	8
5.2 CABLE ATTACHMENT PROCEDURE	9
5.3 FITTING APPLANATION TONOMETERS, T TYPE AND R TYPES.....	10
6. INSTRUCTIONS FOR USE	11
6.1 SETTING THE BINOCULARS.....	11
6.2 PREPARING THE PATIENT AND USING THE SLIT LAMP	12
6.3 DESCRIPTION OF FILTERS, APERTURES AND MAGNIFICATIONS.....	13
7. ROUTINE MAINTENANCE	14
7.1 LED SYSTEMS.....	14
7.2 REGULARLY INSPECT THE DEVICE FOR DAMAGE OR DIRT.....	14
7.3 CLEANING AND CHANGING THE MIRROR.....	14
7.4 ELECTRICAL CONNECTIONS	14
7.5 OPTICS.....	14
7.6 AXLE AND MECHANICAL PARTS	14
8. WARRANTY	15
9. SPECIFICATIONS AND ELECTRICAL RATINGS.....	15
9.1 ELECTROMAGNETIC EMISSIONS.....	15
9.2 ELECTROMAGNETIC IMMUNITY	16
9.3 RECOMMENDED SAFE DISTANCES	17
9.4 TECHNICAL SPECIFICATIONS	18
10. ACCESSORIES AND SPARES	20
11. PACKAGING AND DISPOSAL INFORMATION.....	21

	Consult instructions for use		General warning sign
	Date of manufacture		Warning: Electricity
	Manufacturer's name and address		Warning: Floor level obstacle
	Country of manufacture		Warning: Non-ionizing radiation
	Waste Electrical and Electronic Equipment (WEEE) recycling		Warning: Optical radiation
	This way up		Warning: Hot surface
	Keep dry		Conformité Européene
	Fragile		Type B applied part
	Do not use if package is damaged		Class II equipment
	Temperature limit		Atmospheric pressure limitation
	Authorised representative in the European Community		Humidity limitation
	Catalogue number		Serial number
	Translation		Medical device

The Rodenstock Slit Lamp is designed and built in conformity with Directive 93/42/EEC, Regulation (EU) 2017/745 and ISO 13485 Medical Devices Quality Management Systems.

Classification: CE: Class I

FDA: Class II

The information contained within this manual must not be reproduced in whole or part without the manufacturer's prior written approval. As part of our policy for continued product development we the manufacturer reserve the right to make changes to specifications and other information contained in this document without prior notice.

This IFU is also available on the Rodenstock website

Copyright © Rodenstock Instruments 2021. Published in the UK 2021.

1. INDICATIONS FOR USE

This device is intended to be used only by suitably trained and authorised healthcare professionals.

 **CAUTION: Federal Law restricts this device to sale by or on the order of a physician or practitioner.**

Intended use / purpose of instrument

This Rodenstock Slit Lamp is an AC-powered slit lamp biomicroscope and is intended for use in eye examination of the anterior eye segment, from the cornea epithelium to the posterior capsule. It is used to aid in the diagnosis of diseases or trauma which affects the structural properties of the anterior eye segment.

Brief description of the instrument


This Rodenstock Slit Lamp can either be mounted onto a custom table top supplied by Rodenstock or can be mounted on a third parties table top (refraction unit) by suitably trained technicians.

The Rodenstock Slit Lamp consists of 4 assemblies; Illumination Tower; Observation System; XYZ Translation Base; Chinrest Assembly

The light intensity is controlled by a variable rheostat located on the XYZ Translation Base. There are a number of selectable filters allowing the user to control the characteristics of the examination light.

2. SAFETY

2.1 PHOTOTOXICITY

 **CAUTION: The light emitted from this instrument is potentially hazardous. The longer the duration of exposure, the greater the risk of ocular damage. Exposure to light from this instrument when operated at maximum intensity will exceed the safety guideline after 171 seconds.**

While no acute optical radiation hazards have been identified for slit lamps, we recommend keeping the intensity of the light reaching the patient's retina to the minimum possible for the respective diagnosis. Children, people with aphakia and people suffering from eye conditions are most at risk. An increased risk may also occur if the retina is exposed to the same or a similar device with a visible light source within 24 hours. This applies, in particular, if the retina has been photographed with a flashbulb in advance.

Keeler Ltd shall on request, provide the user with a graph showing the relative spectral output of the instrument.

2.2 WARNINGS AND CAUTIONS

Please note that the proper and safe functioning of our instruments is only guaranteed if both the instruments and their accessories are exclusively from Rodenstock Instruments. The use of other accessories may result in increased electromagnetic emissions or reduced electromagnetic immunity of the device and may lead to incorrect operation.

Observe the following precautions in order to ensure safe operation of the instrument.

WARNINGS

- Never use the instrument if visibly damaged and periodically inspect it for signs of damage or misuse.
- Check your Rodenstock product for signs of transport / storage damage prior to use.
- Do not use in the presence of flammable gases / liquids, or in an oxygen rich environment.
- US Federal Law restricts this device to sale by or on the order of a physician or practitioner.
- This device is intended to be used only by suitably trained and authorised healthcare professionals.
- This product should not be immersed in fluid.
- Repairs and modifications to the instrument must be made only by the specialized technicians of the manufacturer's Technical Service Centre or by personnel trained and authorised by the manufacturer. The manufacturer declines any and all responsibility for loss and / or damages resulting from unauthorised repairs; furthermore, any such actions will invalidate the warranty.
- The power switch and mains plug are the means of isolating the device from the mains supply – ensure both the power switch and mains plug are accessible at all times.
- Do not position the equipment so that is difficult to press the power switch or remove the mains plug from the wall socket.
- Refraction stand variants or adaptors should only be used in combination with EN/IEC 60601-1 and EN/IEC 60601-1-2 compliant power supplies and devices.



- Route power cords safely to eliminate risk of tripping or damage to user.



- Before any cleaning of the instrument or the base unit ensure the power lead is disconnected.



- LEDs can reach high temperatures in use – allow to cool before handling.



- Do not exceed maximum recommended exposure time.

- Should the instrument suffer shocks (for example, should it accidentally fall), and the optical system or the illumination system are damaged it may be necessary to return the instrument to the manufacturer for repair.

- After removal of the LED, do not touch the Slit Lamp LED electrical contacts and patient simultaneously.
- The owner of the instrument is responsible for training personnel in its correct use.
- Ensure the instrument or instrument table is placed on a level and stable surface.
- Use only genuine Rodenstock approved parts and accessories or device safety and performance may be compromised.
- Shut down after every use. In case the dust cover is used: risk of overheating.
- For indoor use only (protect from moisture).
- Electrical equipment can be affected by electromagnetic interference. If this occurs whilst using this equipment, switch the unit off and reposition.
- Do not touch accessible connectors and the patient simultaneously.



Before use, the Slit Lamp should be allowed to adjust to the ambient room temperature for several hours. This is especially important when the unit has been stored or transported in a cold environment; this can cause severe condensation to develop on the optical elements.

2.3 CONTRAINDICATION

There is no restriction to patient population this device can be used with other than those outlined in the contra-indications stated below.

Slit lamps can produce discomfort in some photophobic patients due to the high illumination. Additionally patients must be co-operative and capable of sitting upright for the duration of the examination, therefore the technique may not be suitable for patients who are unable to sit upright for long periods of time or those with limited neck and back mobility.

3. CLEANING AND DISINFECTION INSTRUCTIONS



Before any cleaning of the instrument or the base unit, ensure the power lead is disconnected.


Only manual non-immersion cleaning as described should be used for this instrument. Do not autoclave or immerse in cleaning fluids. Always disconnect power supply from source before cleaning.

1. Wipe the external surface with a clean absorbent, non-shedding cloth dampened with de-ionised water / detergent solution (2% detergent by volume) or water / isopropyl alcohol solution (70% IPA by volume). Avoid optical surfaces.
2. Ensure that excess solution does not enter the instrument. Use caution to ensure cloth is not saturated with solution.
3. Surfaces must be carefully hand-dried using a clean non-shedding cloth.
4. Safely dispose of used cleaning materials.

4. NAMES OF CONTROLS AND COMPONENTS


Headrest Assembly

1. Fixation light cable
2. Main lamp cable (4 pin socket)
3. Forehead rest band
4. Patient's eye height marker
5. Fixation light
6. Chinrest
7. Chinrest height adjuster
8. Patient grab handles
9. Power lead, chinrest to lamp housing

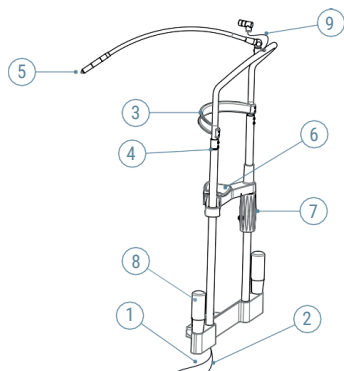


RSL 5000 Series



10. Lamp cover
11. Lamp cover release screws
12. Lever for grey (ND), blue, diffuser and red free filters
13. Slit length, slit rotation and aperture control
14. Scale for slit rotation
15. Illumination mirror
16. Slit offset centering knob
17. Inclination latch 5° to 20°
18. Slit width controls
19. Test bar & tonometer plate mounting hole and cover
20. Illumination arm locking knob
21. Microscope arm locking knob

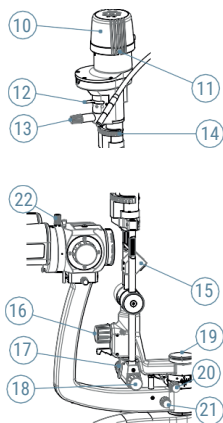

22. Eyepiece assembly securing knob
23. Joystick base locking knob
24. Joystick control (X Y Z movement)
25. Illumination control rheostat
26. Axle
27. Runner covers

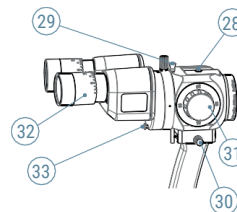

28. R type tonometer mounting hole
29. Yellow filter knob (up = out)
30. Lock for securing magnification body
31. Magnification change drum
32. Eyepieces – adjustable for PD and dioptre correction
33. Breath shield securing knob
34. Main power switch
35. Power supply unit
36. Glide plate

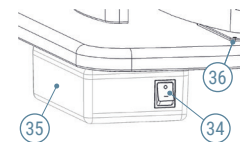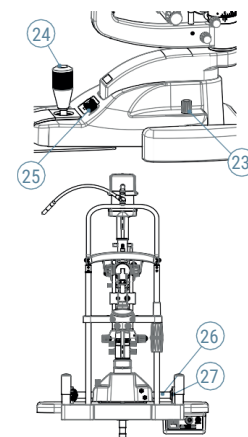

5. ASSEMBLY

Your Rodenstock Slit Lamp has been designed to fit on to an electric insulated medical table base or on to an electric insulated and fire resistant medical table top, e.g. a Refraction Stand or Combi Unit.

Take care when unpacking your Slit Lamp that you do not accidentally damage or discard any of the contents.



Leave the Slit Lamp in the packing for several hours after delivery before unpacking to reduce the risk of condensation forming.

Rodenstock Slit Lamps can be fitted to most Refraction Stands / Combi Units. Rodenstock advises that this shall be carried out by suitably trained technicians to ensure performance and safety are not affected.



The Refraction Stand, Combi Unit or table leg must be IEC 60601-1 compliant.


If you are fitting or have fitted your Slit Lamp to a medical or Rodenstock table leg / base ensure it is situated on a firm and level floor.

If the table leg / base has castors ensure the following before moving it to another location:


1. The table is at its lowest position.
2. The power cord is removed.
3. The Slit Lamp arm and base locking knobs are tightened.
4. The runner covers are securely located.
5. The system is moved by grasping it at its lowest practical point.

5.1 ILLUMINATION TOWER ASSEMBLY PROCEDURE


1. Remove the hex bolt (a) from the base of the Illumination tower, and then place the illumination tower on the Slit Lamp base with the base notch (b) and pin (c) aligned. Attach the tower to the base using the hex bolt removed earlier and tighten using the wrench provided.

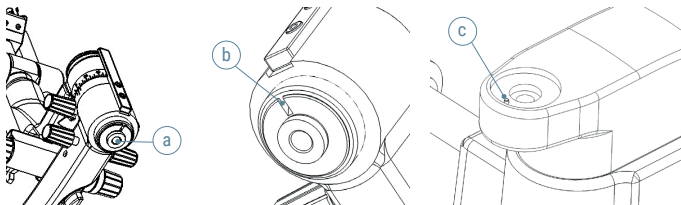

2. Carefully fit the microscope body to the arm – ensuring it is pushed to the stop. Tighten using the securing knob on the side.

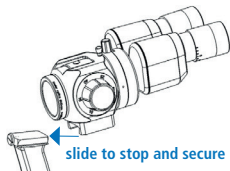

3. Attach the breath shield as indicated in the images below.


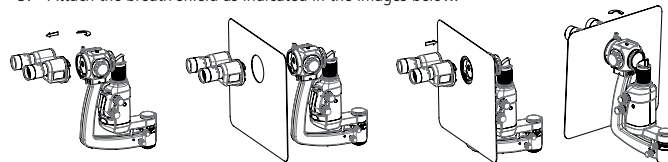


Note: Images show assembly on RSL 4300/4500, however the process is identical and applicable to RSL 5000.

5.2 CABLE ATTACHMENT PROCEDURE


1. Connect the main lamp cable from the chinrest to the illumination tower. Do not twist the lead on the tower illumination system.
2. Connect the power cables.
 - a) Chinrest fixation light cable to power supply unit.
 - b) (3 pin) cable from power supply unit to slit lamp base assembly.
 - c) (4 pin) main lamp cable from bottom of the chinrest to the slit lamp base assembly.
 - d) Ensure cables are routed to allow free movement of the XYZ base and to be clear of the patients.

Cable connector

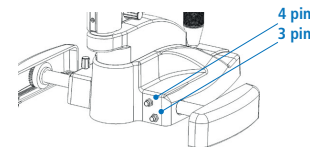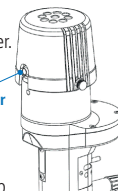

If your Slit Lamp was not supplied with a transformer (Part #3020-P-5040-ROD), make sure that the power connection is compatible with the specifications in this manual and is connected by a qualified technician to an available and suitable power supply, see section 9.4 Power Supply page 19.

3. Connect the mains power to the Slit Lamp transformer using the power lead provided.

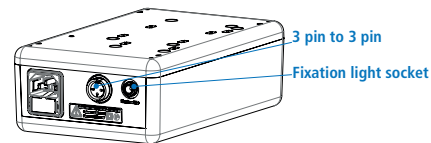



Only a hospital grade 3-conductor electrical power supply cable must be used. For USA and Canada: Detachable power supply cord set, UL listed, type SJE, SJT or SJO, 3-conductor, not smaller than 18 AWG. Plug, cable and ground lead connection of the socket have to be in perfect condition.


5.3 FITTING APPLANATION TONOMETERS, T TYPE AND R TYPES

Rodenstock Applanation Tonometer (T-TYPE)

1. Position the guide plate in the tonometer / test bar support hole on the slit lamp.
2. Lift the tonometer out of the packaging and assemble it by inserting the pin on its base into one of the two possible openings (for right or left eye) on the horizontal guide plate above the slit lamp axis. These positions are related to the microscope optics and observation can be made either through the right or the left eye-piece.

The tonometer will slip easily onto the support plate; stability is assured by the locking pins.

3. To obtain an image as clear and as free of reflexes as possible, the angle between the illumination and the microscope should be about 60° and the slit diaphragm should be fully opened.
4. When not in use the tonometer should be removed from the Slit Lamp and placed securely back in the packaging or a suitable location.

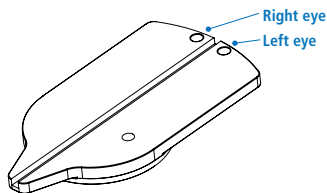

Applanation Tonometer 'Rodenstock Fixed' (R-TYPE)

This instrument is for those who wish the tonometer to remain permanently on the slit lamp.

5. Mount the plate for the tonometer onto the microscope body using the securing screw.
6. Mount the tonometer mount onto the mounting post.
7. Swing the tonometer forward in front of the microscope for examination. A notch position ensures exact centering of the prism with the left objective.
8. To obtain an image as clear and as free of reflexes as possible, the angle between the illumination and the microscope should be about 60° and the slit diaphragm should be fully opened.
9. When not in use the instrument is swung around and secured in a notch position to the right of the microscope.


6. INSTRUCTIONS FOR USE

6.1 SETTING THE BINOCULARS




It is vital that the binoculars are optimised for the user's optical correction in order to obtain focused binocular images.

1. Remove the Test Bar locating hole cover plate (19) and place the test bar focus in the test bar location hole at the base of the microscope arm. To access the location hole first remove the cover. The test bar should be set with the flat projection face towards the Slit Lamp Microscope. The illumination and the microscope should be in the zero degrees position.
2. Turn on the Slit Lamp and set the slit to full width (18), set the magnification to x16 (31).
3. Adjust the eyepieces pupillary distance by holding both eyepiece bodies and rotating them inwards or outwards until they are correct for your PD.

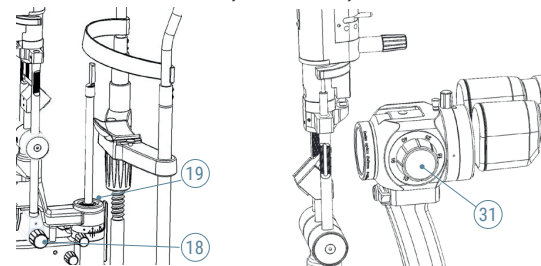

4. Turn both eyepieces (32) to maximum plus (+) correction.
5. Close one eye, and with the other eye look through the microscope slowly turning the open eye eyepiece towards the minus (-) position until the image of the test bar is in focus. Stop.
6. Repeat the above process for the other eyepiece.
7. Make a note of the positions of the eyepieces so that you can set them quickly if the Slit Lamp has been used by another clinician.
8. Note – younger examiners are recommended to compensate for their ability to accommodate by further adjusting the eyepieces by minus one (-1) or minus two (-2) dioptres.

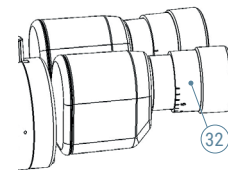


6.2 PREPARING THE PATIENT AND USING THE SLIT LAMP

! Parts of the slit lamp coming in to contact with the patient should be cleaned in accordance with these instructions prior to the examination. Rodenstock recommends the use of disposable hygienic chinrest tissues on the chinrest before the patients place their chin on it.

Never use the instrument if visibly damaged and periodically inspect it for signs of damage or misuse.

1. The patient should be as comfortable as possible and with the patient in the chinrest adjust the chinrest height (7) so that the patient's eyes are level with the height marking (4) on the chinrest support.
2. Focus the eyepieces using the test bar as described earlier, and if you have not already done so set them to your interpupillary distance by holding both eyepiece bodies and rotating them inwards or outwards until they are correct for your PD.
3. Switch on the illumination, making sure the rheostat (25) is set to a low level to minimise the patient's exposure to light hazard.
4. Rotate the joystick (24) until the light beam is at eye level.
5. Holding the joystick vertical, move the slit lamp base towards the patient until the slit beam appears focused on the patient's cornea.
6. Adjust slit width (18), magnification (31), slit rotation (13) & slit angle etc. as required to perform the examination.
7. Loosening slit offset centering knob (16) to allow the slit image to be moved off centre for scleral illumination. Tightening the knob will re-centre the slit image in the centre of the visual field of the microscope.

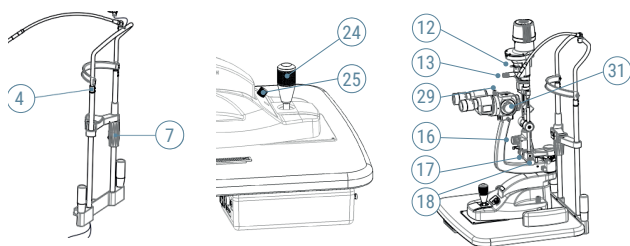

8. The slit image is made vertical, or given a pre-set angle by means of the inclination latch (17) (notches at 5°, 10° and 15° & 20°).
9. When using the blue filter (12) the user may wish to insert the yellow barrier filter (29). The yellow barrier filter is out when the knob is up, in when it is down.

10. When the examination is complete, set the rheostat to a low level and switch off the Slit Lamp.

! Shut down after every use. In case the dust cover is used: risk of overheating.

6.3 DESCRIPTION OF FILTERS, APERTURES AND MAGNIFICATIONS

Stereo microscope

Eyepieces	12.5x
Dioptic adjustment	+/- 8D
PD range	49mm-77mm
Convergent angle of optical axis	13°

5 Step magnification change

Magnification	Field of view
x6	34mm
x10	22mm
x16	14mm
x25	8.5mm
x40	5.5mm

Filters

- a) Clear
- b) Neutral Density
- c) Red Free
- d) Diffuser
- e) Blue



Apertures



Aperture diameters (mm)

Tower illumination

Tower has the facility to tilt towards the user and positively locates at each step. 0°, 5°, 10°, 15° and 20°.

7. ROUTINE MAINTENANCE

⚠ The maintenance outlined below should only be carried out once the main power cable has been disconnected. If you have any problems that are not covered by the procedures described below, contact Rodenstock or your local supplier.

7.1 LED SYSTEMS

LED's typically have a life exceeding 10,000 hours of continuous use and therefore can be considered as a non-consumable item that will not require changing by a user.

Whilst this is a significant life expectancy, we suggest that the Slit Lamp is always switched off between examinations to conserve energy and LED life.

In the unlikely event of an LED failure please contact Rodenstock or your local distributor for guidance on the replacement procedure.

7.2 REGULARLY INSPECT THE DEVICE FOR DAMAGE OR DIRT

Routinely clean as per section 3 on page 5 cleaning instructions.

7.3 CLEANING AND CHANGING THE MIRROR

The mirror is front surfaced to avoid ghosting of the projected light and therefore is very delicate and will need to be replaced when the surface deteriorates.

The mirror should be cleaned only with a soft clean lens cloth.

The mirror is an interference fit in its holder and can be removed by grasping it firmly and pulling it out of its holder. Slide in the replacement mirror taking care to avoid touching the reflective surface.

Care must be taken to keep the objective and the eyepiece lenses clean – use only soft clean lens cloths to clean optical surfaces.

7.4 ELECTRICAL CONNECTIONS

Routinely check all electrical connections, cables and connectors. To access the bulb connections see earlier in this section for guidance.


7.5 OPTICS

The optics should be wiped clean of any loose dirt or debris with a suitable dust brush then cleaned with a soft dry lens cloth, washed linen or other non abrasive lens cleaning material.

The condenser lens beneath the illumination lamp will require cleaning; to access this, remove the lamp as detailed earlier, clean the condenser lens and then replace the lamp.

7.6 AXLE AND MECHANICAL PARTS

If the Slit Lamp becomes hard to move on the glide plate, the plate should be cleaned with a lightly oiled cloth or silicon polish. The axle should be cleaned only with dry lint free cloths.

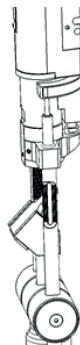

8. WARRANTY

The Rodenstock Slit Lamps are guaranteed for two years against faulty workmanship materials or factory assembly. Warranty is on a Return To Base (RTB) basis at the cost of the customer and may be void if the Slit Lamp has not been regularly serviced.

The manufacturer's warranty and terms and conditions are detailed on the Rodenstock website.

The mirror, main illumination lamp and general 'wear and tear' are excluded from our standard warranty.

⚠ The manufacturer declines any and all responsibility and warranty coverage should the instrument be tampered with in any manner or should routine maintenance be omitted or performed in manners not in accordance with these manufacturer's instructions.

There are no user serviceable parts in this instrument. Any servicing or repairs should only be carried out by Rodenstock Instruments or by suitably trained and authorised distributors. Service manuals will be available to authorised Rodenstock service centres and Rodenstock trained service personnel.

9. SPECIFICATIONS AND ELECTRICAL RATINGS

The Rodenstock Slit Lamp is a medical electrical instrument. The instrument requires special care concerning electromagnetic compatibility (EMC). This Section describes its suitability in terms of electromagnetic compatibility of this instrument. When installing or using this instrument, please read carefully and observe what is described here.

Portable or mobile-type radio frequency communication units may have an adverse effect on this instrument, resulting in malfunctioning.

9.1 ELECTROMAGNETIC EMISSIONS

Guidance and manufacturer's declaration – electromagnetic emissions

The Rodenstock Slit Lamp is intended for use in the electromagnetic environment specified below. The customer or user should ensure that it is used in such an environment.

Emissions test	Compliance	Electromagnetic environment – guidance
RF emissions CISPR 11	Group 1	The Rodenstock Slit Lamp uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	The Rodenstock Slit lamp is suitable for use in a professional healthcare facility environment. The Rodenstock Slit lamp is not intended for use in home environment.
Harmonic emissions IEC 61000-3-2	Class B	

Emissions test	Compliance	Electromagnetic environment – guidance
Voltage fluctuations / flicker emissions IEC 61000-3-3	Complies	


9.2 ELECTROMAGNETIC IMMUNITY

Guidance and manufacturer's declaration – electromagnetic immunity

The Rodenstock Slit Lamp is intended for use in the electromagnetic environment specified below. The customer or user should ensure that it is used in such an environment.

Immunity test	IEC 60601 Test level	Compliance level	Electromagnetic environment – guidance
Electrostatic discharge (ESD). IEC 61000-4-2	± 8 kV contact ± 15 kV air	± 8 kV contact ± 15 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Electrical fast transient / burst. IEC 61000-4-4	± 2 kV for power supply lines ± 1 kV for power supply lines	± 2 kV for power supply lines ± 1 kV for power supply lines	Mains power quality should be that of a typical professional healthcare facility
Surge. IEC 61000-4-5	± 1 kV line(s) to line(s) ± 2 kV line(s) for input / output line(s)	± 1 kV line(s) to line(s) ± 2 kV line(s) for input / output line(s)	Mains power quality should be that of a typical professional healthcare facility
Voltage dips, short interruptions and voltage variations on power supply input lines. IEC 61000-4-11	$U_T = 0\%$ 0.5 cycle (0, 45, 90, 135, 180, 225, 270, 315°) $U_T = 0\%$; 1 cycle $U_T = 70\%$; 25 / 30 cycles (@ 0°) $U_T = 0\%$; 250 / 300 cycle	$U_T = 0\%$ 0.5 cycle (0, 45, 90, 135, 180, 225, 270, 315°) $U_T = 0\%$; 1 cycle $U_T = 70\%$; 25 / 30 cycles (@ 0°) $U_T = 0\%$; 250 / 300 cycle	Mains power quality should be that of a typical professional healthcare facility environment. If the user of the Rodenstock Slit Lamp requires continued operations during power mains interruptions, it is recommended that the instrument be powered from an uninterruptible power supply.
Power frequency (50 / 60 Hz) Magnetic field. IEC 61000-4-8	30 A/m	30 A/m	Power frequency magnetic fields should be at a level characteristic of a typical location in a typical professional healthcare facility environment.

Note: U_T is the a. c. mains voltage prior to application of the test level.

Immunity test	IEC 60601 Test level	Compliance level	Electromagnetic environment – guidance
			Portable and mobile RF communications equipment should be used no closer to any part of the Rodenstock Slit Lamp, including cables, than the recommended separation distances calculated from the equation applicable to the frequency of the transmitter.
			Recommended separation distance
Conducted RF IEC 61000-4-6	6 Vrms	6 V	$d = 1.2 \sqrt{p}$
Radiated RF IEC 61000-4-3	10 V/m 80MHz to 2.7GHz	10 V/m	$d = 1.2 \sqrt{p}$ 80MHz to 800 MHz $d = 2.3 \sqrt{p}$ 800MHz to 2.7GHz
			Where p is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in metres(m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey ¹ , should be less than the compliance level in each frequency range. ²  Interference may occur in the vicinity of equipment marked with this symbol.

Note: At 80MHz and 800MHz, the higher frequency range applies. These guide lines may not apply in all situations.

Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

1 Field strengths from fixed transmitters, such as base stations (cellular / cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the Rodenstock Slit Lamp is used exceeds the applicable RF compliance level above, the Rodenstock Slit Lamp should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orientating or relocating the Rodenstock Slit Lamp.

2 Over the frequency range 150kHz to 80 MHz, field strengths should be less than 10 V/m.

9.3 RECOMMENDED SAFE DISTANCES

Recommended separation distances between portable and mobile RF communications equipment and the Rodenstock Slit Lamp

The Rodenstock Slit Lamp is intended for the use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the Rodenstock Slit Lamp can help prevent electromagnetic interference by maintaining a minimum distance between mobile RF communications equipment (transmitters) and the Rodenstock Slit Lamp

as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter (W)	Separation distance according to frequency of transmitter (m)		
	150 kHz to 80MHz $d = 1.2\sqrt{p}$	80MHz to 800MHz $d = 1.2\sqrt{p}$	800MHz to 2.7GHz $d = 2.3\sqrt{p}$
0.01	0.12	0.12	0.23
0.1	0.38	0.38	0.73
1	1.2	1.2	2.3
10	3.8	3.8	7.3
100	12	12	23

For transmitters rated at a maximum output power not listed above, the recommended separation distance in metres (m) can be determined using the equation applicable to the frequency of the transmitter, where p is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

Note: At 80MHz and 800MHz, the separation distance for the higher frequency applies.

These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

9.4 TECHNICAL SPECIFICATIONS

Optical System

Type	Galilean converging binoculars @ 8°
Magnification	Rotating drum change x6, x10, x16, x25 & x40
Eyepiece	x12.5
Field of view	34, 22, 14, 8.5 and 5.5 mm
Interpupillary distance	49.0 to 77mm
Objective lens focal distance	107mm
Objective lens convergence angle	13°

Slit Projection System & Base

Slit Width	0-12mm continuously variable
Slit Length	12mm (1mm-12mm continuously variable)
Slit Lamp with chinrest	20.0Kg, 75 x 54 x 45cm W x D x H
Table top with PSU & Acc drawer	5.2Kg, 51 x 42 x 15cm W x D x H
Aperture diameters	0.2, 1mm square, 2, 3, 5, 9 and 12mm

Filters	Clear; red free; neutral density; diffuser; blue; IR heat absorbing filter permanently installed
Slit angle	+/- 90° continuous
Slit rotation	+/- 180° with reference scale
Slit vertical Tilt	0°, 5°, 10°, 15° & 20°
Base travel	25mm Z-axis, 107mm X-axis, 110mm Y axis
Horizontal fine adjustment	12mm
Table top dimensions	405 x 500mm
Fixation lamp	LED
Light source	LED
LED output power	160KLux (+/-20%)

Weight, packed (approx.)

Slit Lamp with chinrest	20.0Kg, 75 x 54 x 45cm W x D x H
Table top with PSU & Acc drawer	5.2Kg, 51 x 42 x 15cm W x D x H

Protection against ingress IPx0

Class II ME equipment

Insulation between mains parts and the functional earth provide at least two means of protection.


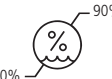
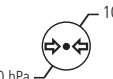
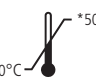

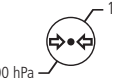



Power Supply

Power supply unit	Switch mode, (100V-240V input) +/- 10% multi plug compliant to EN / IEC 60601-1 EN / IEC 61000-6-2, EN / IEC 61000-6-3
Fuse	T2.5AH, 250V
Power supply output	12V DC: 2.5 amps must be EN / IEC 60601 compliant
Complies with	Electrical Safety (Medical) EN / IEC 60601-1 Electromagnetic compatibility EN / IEC 60601-1-2 Ophthalmic instruments - Fundamental requirements and test methods ISO 15004-1 Ophthalmic instruments - Optical radiation hazard ISO 15004-2

Fuse ratings and quantity

2.5 amp anti-surge
Fuse current 2.5A
Voltage rating V AC 250V
Breaking capacity 1500Amps
Blow characteristic: Time Delay

Environmental Conditions:

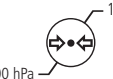USE	
	
	
Shock (without packing)	10 g, duration 6 ms
STORAGE CONDITIONS	
	
	
TRANSPORT CONDITIONS	
	
	
Vibration, sinusoidal	10 Hz to 500 Hz: 0.5g
Shock	30 g, duration 6 ms
Bump	10 g, duration 6 ms

*This instrument does not meet the temperature requirements of ISO 15004-1 for storage and transportation. Do not store or transport this instrument in conditions where the temperature may rise above 50 °C.

10. ACCESSORIES AND SPARES

Item	Part Number
1.27mm A/F ALLEN KEY	EP79-00766
1m DSL to PSU Cable	3020-P-7068
5mm L-Shaped Hexagonal Key	EP79-10892
6mm L-Shaped Hexagonal Key	EP79-10905
Black Patient Handle	EP79-80070
Cable Clip	EP39-80329
Chin Rest Pins (406)	EP39-80321
Digital Base to Computer Cable	3020-P-7077

Item	Part Number
DSL Accessory Kit Packaging	3020-P-5221
DSL Digital Background Illum.Assy	3020-P-5039
DSL Digital Backlight Filters	3020-P-7054
DSL DIGITAL LEFT/RIGHT LABEL	EP39-80365
DSL DIGITAL X-Y PAD	EP39-80364
DSL Illum Projection Mirror-Small	EP39-80250
DSL Refraction Stand Cable Kit	3020-P-7014
DSL Screw Kit	3020-P-5240
DSL Table Template	EP39-80324
Dust Cover - No branding	EP39-19072
Fixation&llum to 6 Pin Connector	3020-P-7117
Linear Gear Rack (405)	EP39-80173
Linear Gear Rack Cover/silver	EP39-80174
Refraction Cable (XY-Bulb)	3020-P-7013
Rodenstock PSU 12V	3020-P-5040-ROD
USB3 Type-A to Micro-B Cable Assy	3020-P-7107
XY Pad (405)	EP39-80169

11. PACKAGING AND DISPOSAL INFORMATION**Disposal of old electrical and electronic equipment**

This symbol on the product or on its packaging and instructions indicates that this product shall not be treated as household waste.

To reduce the environmental impact of WEEE (Waste Electrical Electronic Equipment) and minimise the volume of WEEE entering landfills we encourage at product end of life that this equipment is recycled and reused.

If you need more information on the collection reuse and recycling then please contact B2B Compliance on 01691 676124 (+44 1691 676124). (UK only). Any serious incident that has occurred in relation to the device must be reported to the manufacturer and the competent authority of your Member State.

Contact



Manufacturer

Keeler Limited
Clewer Hill Road



Windsor
Berkshire
SL4 4AA UK

Freephone 0800 521251

Tel +44 (0) 1753 857177

Fax +44 (0) 1753 827145

SUPPLIER

Rodenstock Instruments
Business Unit of Tomey GmbH
Wiesbadener Straße 21
90427 Nürnberg Germany

Tel +49 9131 777 1 777

Fax +49 9131 777 1 20



Visiometrics, S. L., Vinyals, 131
08221 Terrassa, Spain

EP59-70031 Issue 6

Date of Issue 12/05/2021



RODENSTOCK